(depending on model)

If present, first remove the locking plate. Remove the bottle: tilt the bottom of the bottle forwards and lift the bottle out. Remove the dispenser from the wall plate: to do this, press the latch on the back wall of the dispenser and slide the dispenser upwards, off the plate.

9.1 Turn the dispenser on its head, so you look into the housing through the bottom (see image 9.1).

9.2 Using your thumb, press the spring-loaded metal pin, located on the upper, back-side of the dispenser, in the direction of the pump. At the same time, use your other hand to carefully unhook and remove the face shield from the front side of the dispenser.

9.3 Depending on the model, carefully press on the counter housing or clip from the inside outwards, to release it from its socket (see image 9.2).

Assemble in reverse order (see image 9.3).

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### E 10. Information for General Care

The Robert Koch-Institute recommends: “Dispensers for lotion and disinfectants have to be easily cleaned and disinfected. Lotion dispensers must be thoroughly cleaned and disinfected whenever cartridges are changed.” [Source: Robert Koch-Institut, recommendations for hand hygiene].

Dosage of salts or chlorinated solutions can lead to the appearance of corrosion and disruption of function in stainless steel products. In rooms where the air contains salts or chlorine (ie. Saunas or swimming pools etc.), corrosion may appear in spite of fabrication with high quality stainless steel.

In order to guarantee proper functioning of the dispenser regular manual cleaning or processing in cleaning-disinfection devices is necessary.

Manual cleaning: cleaning of the dispenser housing and the pump with hot water. Rinsing the pump with hot water.

Processing in cleaning-disinfection devices: choose program „Dispenser cleaning“ and use pH-neutral cleaning solutions. Important: Cleaning of plastic pumps up to max. 65 °C! Before cleaning or processing in cleaning-disinfection devices the counter must be removed (see 9).

Processing in autoclave: the dispenser and stainless steel pump are autoclavable with superheated steam (134 °C, 2 bar, max. 20 min dwell time). Plastic wall plate, bottle, plastic pumps (K pumps) and counter are not autoclavable.

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### E 11. Use of the “OPHARDT HMS” software (only with ingo-man® plus Counter)

The “Ophardt HMS” (Ophardt Hygiene Monitoring System™) software solution is exclusively available for users of the ingo-man® plus Counter. Within this software functions are combined to allow for convenient acquisition, analysis and archiving of the received counter data. For further information see the software instruction manual.

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

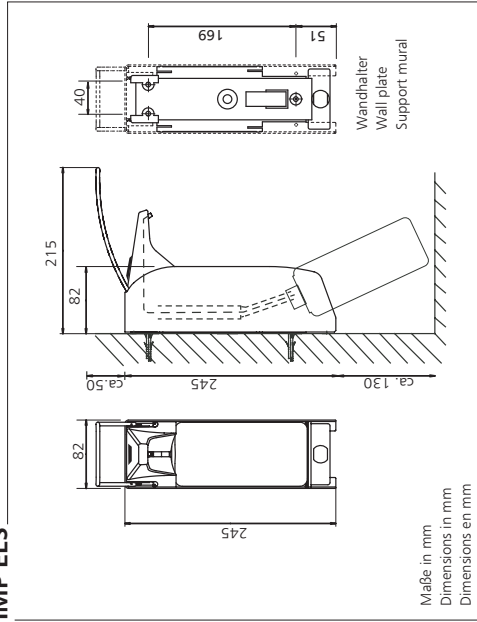
**Important!**  
Please carefully read the operating instructions and save them for reference. The proper function of this dispenser system can only be guaranteed if the procedures in these instructions are followed.

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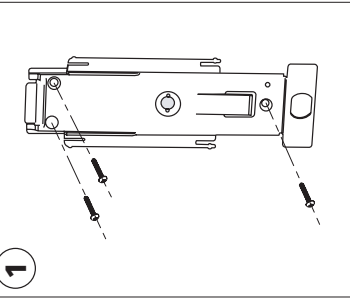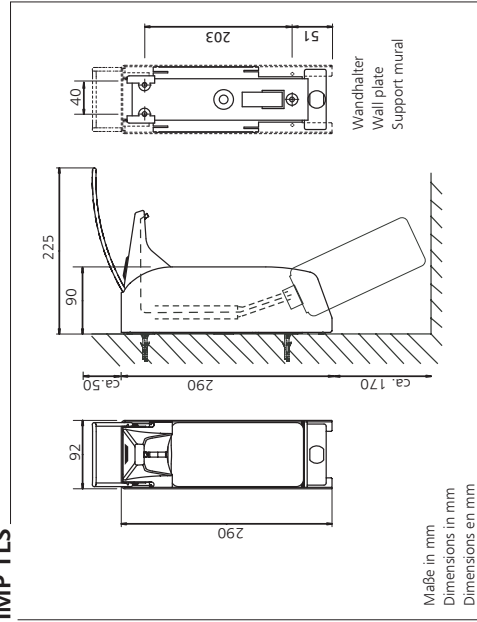
**E 1. Preparation**  
When using as a soap dispenser, install the dispenser as close as possible to the sink to prevent wet hands from dripping onto the floor.  
Required space above the dispenser: 50-70 mm for the lever, and for dispenser removal and placement, as well as an additional 100 mm for operation. The required room below the dispenser, necessary for bottle removal, is about 130 for the IMP E/ELS and 170 mm for the IMP T/TLS.  
Optimal mounting height for use: 130-135 cm, measured from the floor to the upper edge of the dispenser housing.

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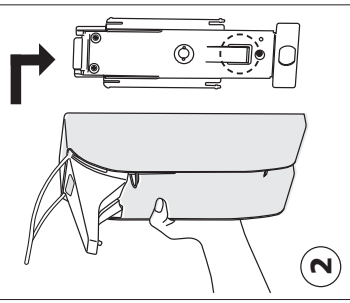**E 2. Mounting of the dispenser**  
First remove the bottle: to do this tilt the bottom of the bottle forwards and then pull the bottle out of the dispenser. Remove the dispenser from the wall plate: press the latch on the back wall of the dispenser and slide the dispenser upwards, off the plate.

**!** Wall plates already in use from the ingo-man® classic/plus product family may continue to be used, if mounting is necessary, please test wall for suitability before mounting.  
With the enclosed, or other suitable attachment materials, attach the wall plate to the wall. Slide the dispenser onto the wall plate from the top until the latch clicks into place (see image 2).

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
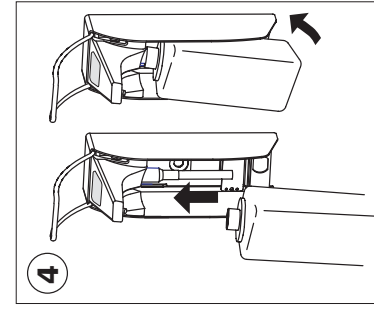
**E 3. Flexible Sealing Cap**  
The included flexible sealing cap ensures optimal sealing for various bottle sizes and prevents the loss of alcohol from the disinfectant solution.

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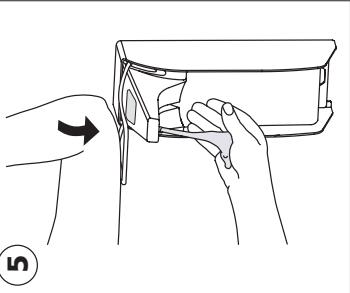**E 4. Bottle Placement / Exchange**  
Insert the filled bottle into the dispenser from the bottom, until snaps into place.  
**F 4. Mise et échange du flacon**

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**E 5. Initial Operation**  
For first use hold your hand under the nozzle and operate the lever several times until solution is dispensed.

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